


Extubation Advisor – 7-Step Guide

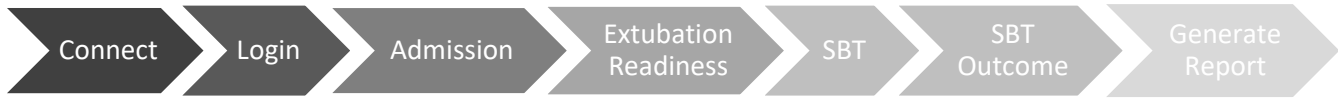


1 Connect

- Connect capnography module to the patient monitor & place EtCO2 cable in-line
- Connect EA tablet / laptop to the patient monitor

2 Login


- Open EA and **login** using a unique 6-digit PIN code

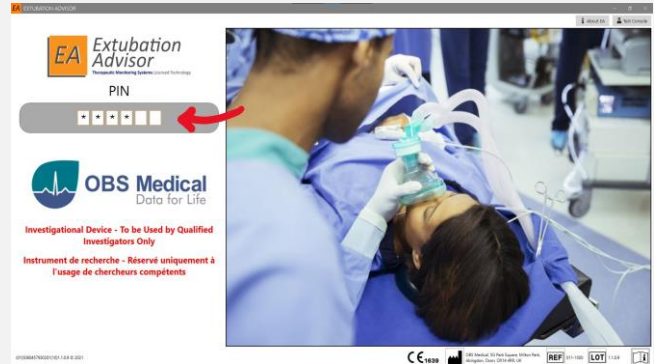

3 Admit New Patient


- To admit a new patient, go to **the Admit New Patient to Roster** section to register the patient
- Once registered, select the patient from the **Patient Roster Selection** section, then click **Perform SBT** to input admission details and relevant comorbidities (one-time data entry)

NOTE: You may proceed to the SBT (and complete the Admission during or post SBT) by selecting SBT tab

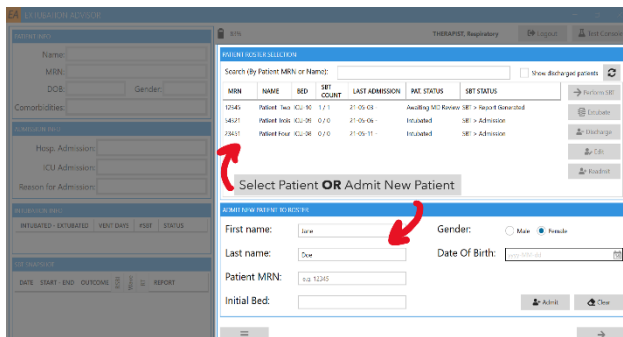


4 Extubation Readiness Checklist

- Complete the **Extubation Readiness** form (can be done pre-, during, or post-SBT)

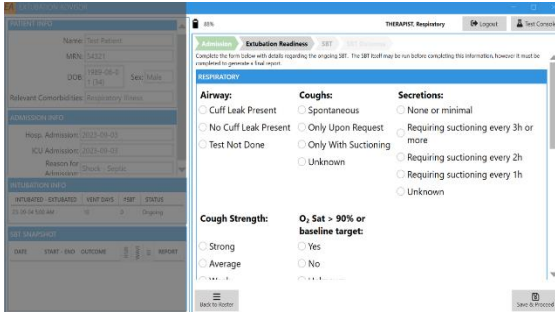

6 SBT Outcome Form


- Select **Save and Proceed** to access the SBT Outcome Form
- Complete SBT Outcome** - input average RR & TV (mL) to calculate RSBI, and provide subjective opinion of the SBT performance
- Once complete, select **Generate Report**

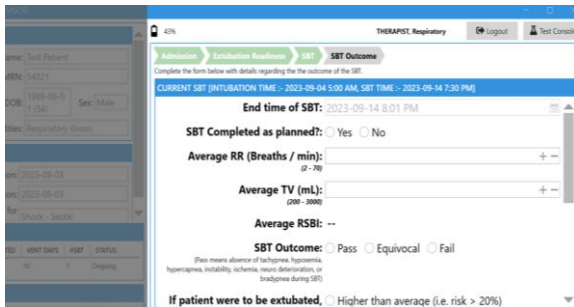

5 Perform SBT

- Document pre-SBT vent settings (PS/PEEP/RASS)
- Verify monitor connections
- Lower vent settings for SBT and **Start Recording**
- Minimum 15mins recording time is required

Once the SBT is done, click **End SBT** and adjust ventilator settings, as per standard practice.

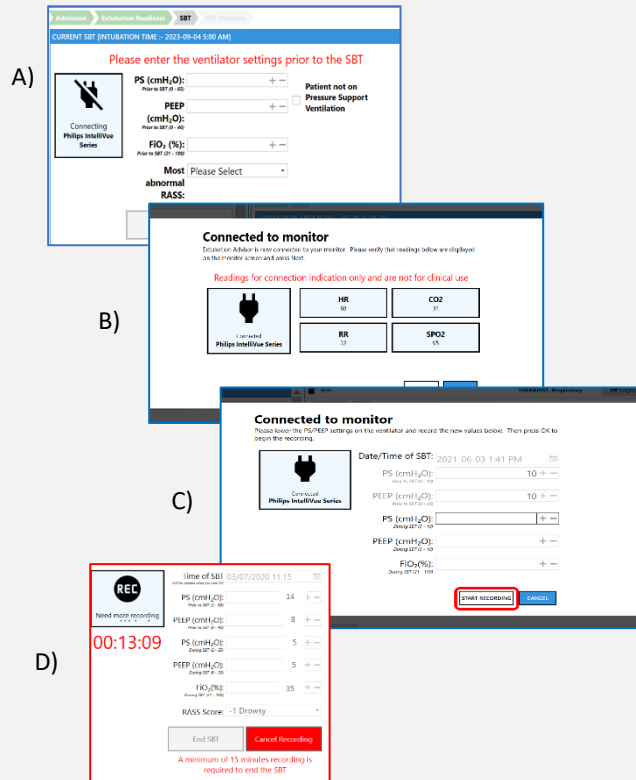

7 Generate SBT Report / Review with MD

- Verify report fields are correct and **Save** the report
- To print the report, select it from the **SBT Snapshot** section (found at the bottom left corner of homepage)
- Review report with the MD prior to extubation decision making
- Log-out, clean device, and return to charging station

